



① Ready for data transmission.




② Open BeneCheck APP and Click the " + " icon to start Bluetooth transmission process.

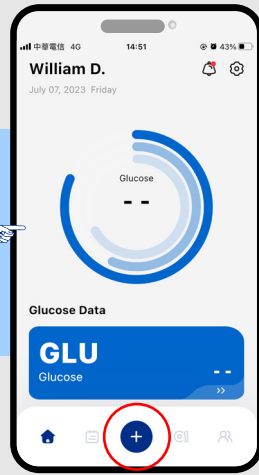


③ Click " Bluetooth " option.

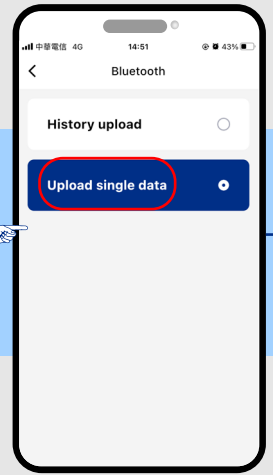


④ Click " Single Data " option.

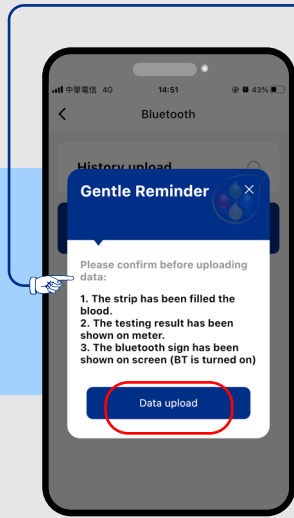


⑤ Check all reminders are completed and then Click " data upload " icon.

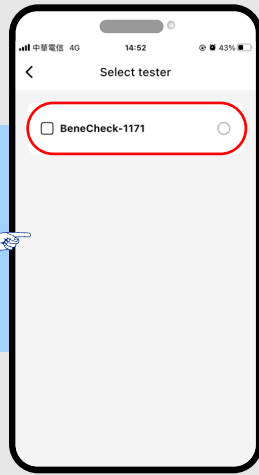


⑥ Search and select your Device ID.

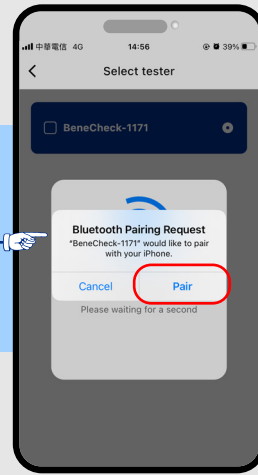


⑦ Pairing device for the first time use.

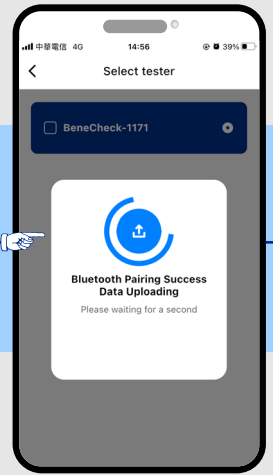


⑧ After pairing is completed, the device will automatically transmit data to APP.

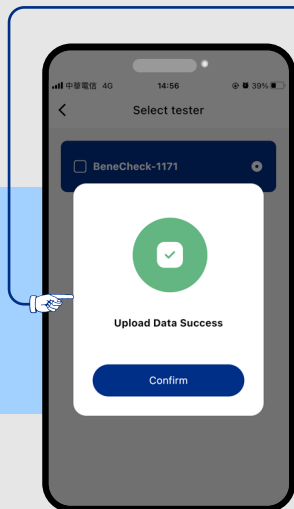


⑨ Complete the transmission and enter the data management page.

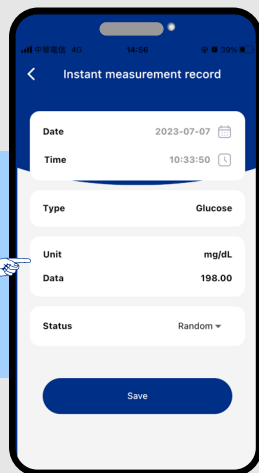


⑩ The management page can modify the meal status. (Only for glucose record).

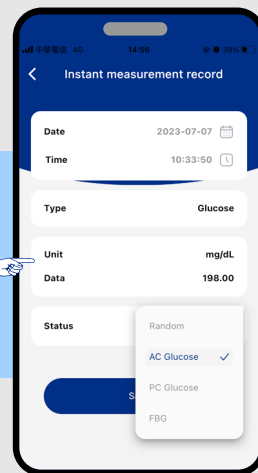


⑪ Save the data and complete the transmission process.

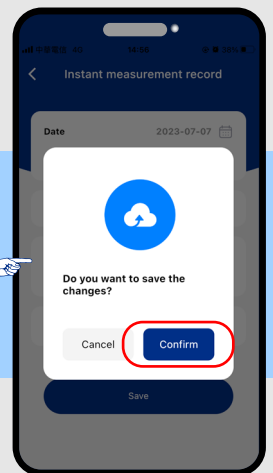


BeneCheck™ Bluetooth historical data upload process

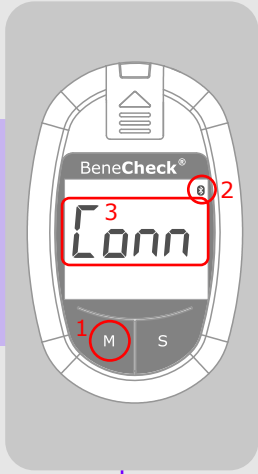


① **In Memory mode:**
When the screen show the "Conn" means ready for data transmission.

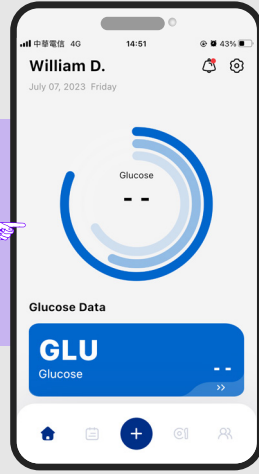


② Open BeneCheck APP.

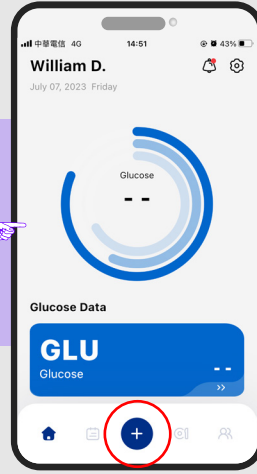


③ Click the "+" icon to start Bluetooth transmission process.

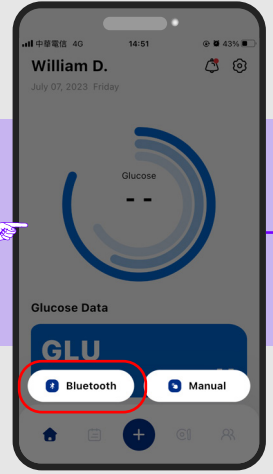


④ Click "Bluetooth" option.

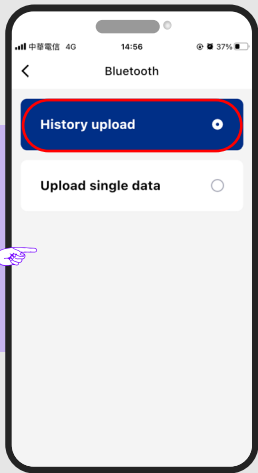


⑤ Click "History upload" option.

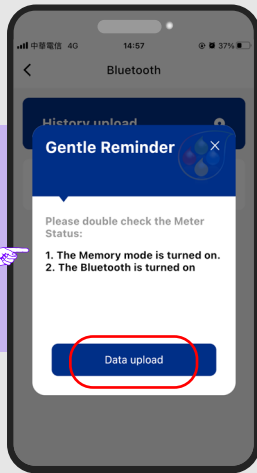



⑥ Check all reminders are completed and then Click "data upload" icon.

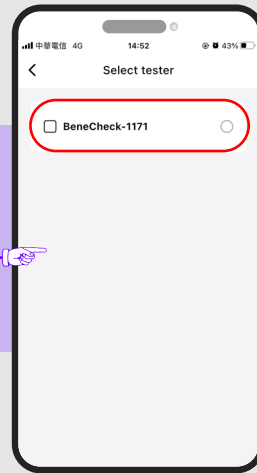


⑦ Search and select your Device ID.

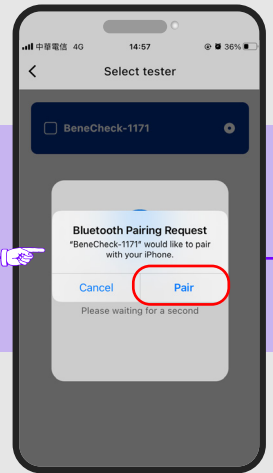

⑧ Pairing device for the first time use.

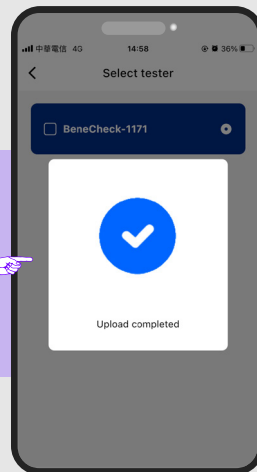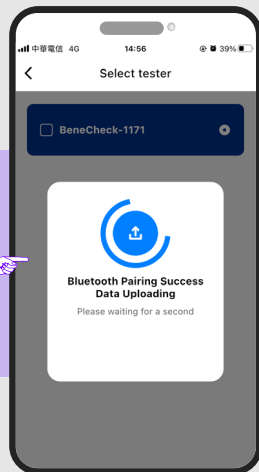

⑨ After pairing is completed, the device will automatically transmit data to APP.

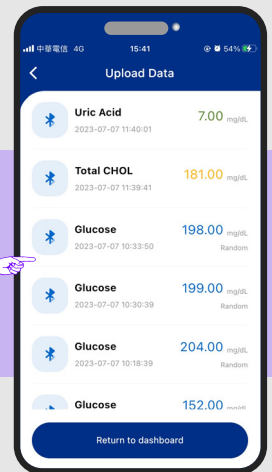

⑩ Complete the transmission and enter the database page.

Download on the App Store

GET IT ON Google Play

